
ACUSHOT NEEDLE FREE Needle-Free Injector

AcuShot Inc is committed to providing next generation needle-free injection technologies to the Chinese marketplace. AcuShot needle-free injection technology has been introduced in the United States, Canada and Europe and has been used in the veterinarian industry.

AcuShot needle-free injectors get their injection power from a proven, more consistent power Cylinder technology. This power Cylinder is compressed and locked into triggering position by our patented motorized mechanism. The motor is driven by the latest in Lithium polymer rechargeable battery technology.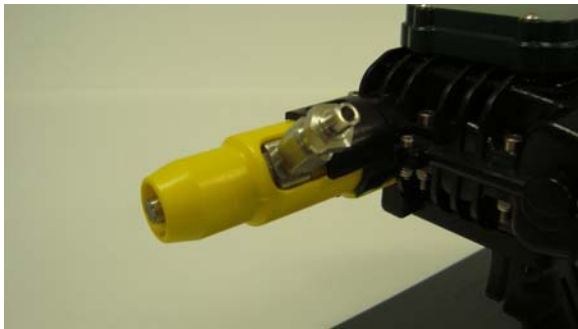

The AcuShot needle-free injectors provide pressure and consistent power delivery during injection. AcuShot needle-free uses power cylinders starting as low as 8000 psi going up to 30,000 psi. The force generated from these power cylinders propels the injection piston forward. The power delivers the needed dosage at the beginning of the injection to pierce into the animal's skin and muscle. The injected dose is absorbed into the muscle and blood stream. The AcuShot A needle-free injector

provides extremely accurate dose at levels range from 0.20 cc – 2.50 cc, adjustable in 0.10 cc increments. The AcuShot S needle-free injector provides extremely accurate dose at micro levels range from 0.05 cc – 1.35 cc, adjustable in 0.05 cc increments. The orifice of each AcuShot needle-free injector is less than 0.7 mm which is three times smaller than the normal needle.

The AcuShot needle-free injectors provide four power cylinders, which can be used on all sizes of livestock.

Power Cylinder Selection				
Pigs / Cows / Sheep		Intradermal	Subcutaneous	Intramuscular
Piglet	Up to 2 weeks	Green	Green	Green
Pig	Up to 60 lbs	Blue	Blue	Blue/Red
Finisher	60 lbs to market weight	Blue/Red	Red	Red/Yellow
Sow		Red/Yellow	Yellow	Yellow
Calves	Up to 6 months	Blue/Red	Blue/Red	Red
Feeder cattle		Red	Red/Yellow	Yellow
Dairy Cows		Blue/Red	Red	Red/Yellow
New born Lamb	Up to two weeks	Green	Green	Green
Lamb	Up to 6 months	Blue	Blue	Blue/Red
Sheep	Over 6 months (Ewe, Ram)	Blue/Red	Red	Red/Yellow

The AcuShot needle-free injector has onboard electronics which monitor key components and injection characteristics for each shot. It helps the operator to simplify priming the injector and select and deliver the accurate dosage. The AcuShot injectors electronically monitor each injection and record data from multiple points. If the data becomes inconsistent or an operating error is made, the injector will make an audible sound and will display the nature of the error on its LCD display.

The recorded data can be transferred to a PC using a USB interface.

The AcuShot Needle-free injectors are battery operated, high workload, handheld, mass vaccination injectors. The battery is removable, rechargeable and installs in the unit. It can be used on a hands-free stand for smaller livestock like new born piglet and poultry or can be used with a remote injection hand piece for easy use with larger livestock.

The AcuShot needle-free injectors are able to perform approximately 400 – 500 injections per battery charge. The power cylinders are able to deliver over 500,000 injections before the power cylinder should be replaced.

The AcuShot needle-free injectors provide the veterinarian industry with revolutionary needle-free equipment that enables spot treatment of single animal and mass-vaccination of large populations in a cost effective, efficient and easy to use format.

The AcuShot needle-free injectors are safer for livestock and reduce chance of infection or disease transmission. It can eliminate the ongoing cost and disposal of hazardous needles. It is also safer for users than needles. The AcuShot injector is a 100% stand alone and self contained unit.

For more information, please contact:

Head office

75 Henlow Bay, Winnipeg, Manitoba, Canada R3Y 1G4

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